



# Aesthetic-Press™:

CE 0535

Zircon

Zircon Bleach

Zircon High Chroma Bleach

Zircon A, Zircon B, Zircon C, Zircon D

## Firing Chart

Press Cycle						
B	↑^	T	N*	H	V1	V2
700	60	840°C	2	20	700°C	840°C

### For Add-On Powder

Firing Temperature							
B	↑^	T	S	H	V1	V2	L
400	45	790°C	6	1	550°C	790°C	700°C

### Glaze Bake (without Glaze Paste)

Firing Temperature							
B	↑^	T	S	H	V1	V2	L
400	45	795°C	4	1	0	0	700°C

\*Firing temperatures can vary. Please check visual results. One may need to adjust temperature.

## Temperature Chart

CELCIUS:	400	550	700	780	900	790	795
FAHRENHEIT:	752	1022	1292	1436	1652	1454	1463

For Ivoclar EP500

### Aesthetic-Press

450 Sutter Street, Suite 1616

San Francisco, CA 94108

Tel. + 1 415 6925372

Fax. + 1 415 723 7075

930
1706